800 T/H

Spin 800

Automatic Biochemistry Analyzer

- PRECISION AND SAFETY IN SAMPLE AND REAGENT HANDLING
- EXPANDABLE SYSTEM
- ADVANCED OPTICAL TECHNOLOGY AND FLEXIBLE CALIBRATION

**Expandable system
up to M1/M2/M3/M4**
(4,800 tests per hour)

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High sampling precision
15~300 μL reagent sampling with step at 0,5 μL , 1,5~35 μL specimen sampling with step at 0,1 μL .

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Water circulation reagent cooling
Ensures a stable 2~8°C refrigeration temperature in reagent disk.

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Direct solid-heating system
Fast heating of reaction disk, ensures temperature is kept at 37°C with 0,1°C fluctuation.

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Effective mixer unit
Independent mixing units for reagents and samples. Three-head mixing bars improve performance. Two-grade washing reduces contaminants significantly.

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Intelligent clot detection
Can detect and differentiate between full-jam, half-jam, and bubbles, ensuring accurate sample aspiration.

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Continuous reagent loading and unloading
Two disk rotation buttons allow safe continuous loading and unloading during testing.

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Throughput: Spin 800: 800T/H, up to 1200T/H with ISE

Sample handling:

Sample tray: 140 positions including 25 cooling positions for calibrators and controls

Sample Module: 300 samples capacity with 30 racks

Sample volume: 1,5~35 µL, step at 0,1µL

Sample probe: Liquid level detection, clot detection and collision protection

Reagent handling:

Reagent tray: 120 positions in coaxial disks

Reagent volume: 15~300 µL, step at 0,5 µL

Sample probe: Liquid level detection, bubble detection and collision protection

Reaction system:

Reaction volume: 100~360 µL

Operating temperature: 37°C with 0,1°C fluctuation

Optical system:

Light source: Halogen tungsten lamp

Photometer: Reversed optics, grating photometry

Wavelength: 340nm, 380nm, 412nm, 450nm, 505nm, 546nm, 570nm, 605nm, 660nm, 700nm, 740nm, 800nm

Absorbance range: 0~3,4Abs(10mm conversion)

ISE Module (Optional):

Principle: Indirect K⁺, Na⁺ Cl⁻, with 22 µL sample aspiration

Control and calibration:

Calibration mode: Linear (one-point, two-point and multi-point), Logit-Log 4P, Logit-Log 5P, Spline,

Exponential, Polynomial, Parabola

Control rules: Westgard multi-rule, Twin plot

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- Vertical and horizontal collision protection for sample and reagent probes.
- Warning log: extensive warning log lists detailed information for operator & service personnel to maintain the system.
- Maintenance guide: easy for operator to perform maintenance procedures, with embedded guides to help users solve problems quickly.

TOYOBO - GROUP

Spinreact SAU
Ctra Sta. Coloma, 7
17176 St. Esteve de Bas
Girona, Spain
Tel. +34 972 690 800
spinreact@spinreact.com

Commercial Office
C/ Aragó, 208, 5o-2a
L'Eixample, 08011
Barcelona, Spain
Tel: +34 93 200 35 44
comercial@spinreact.com

www.spinreact.com

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Tel. +34 854 53 63 74
satcliente@lrdiagnostico.com
www.lrdiagnostico.com